




SMART PRODUCTS
FOR ENDO LOVERS



EssenSeal®

Tea Tree  Thymol  Eugenol 
Root Canal Sealer


The flow test evaluates the cement's fluidity, ensuring it spreads effectively within the root canal, fully penetrating and adapting to the intricate root canal anatomy.

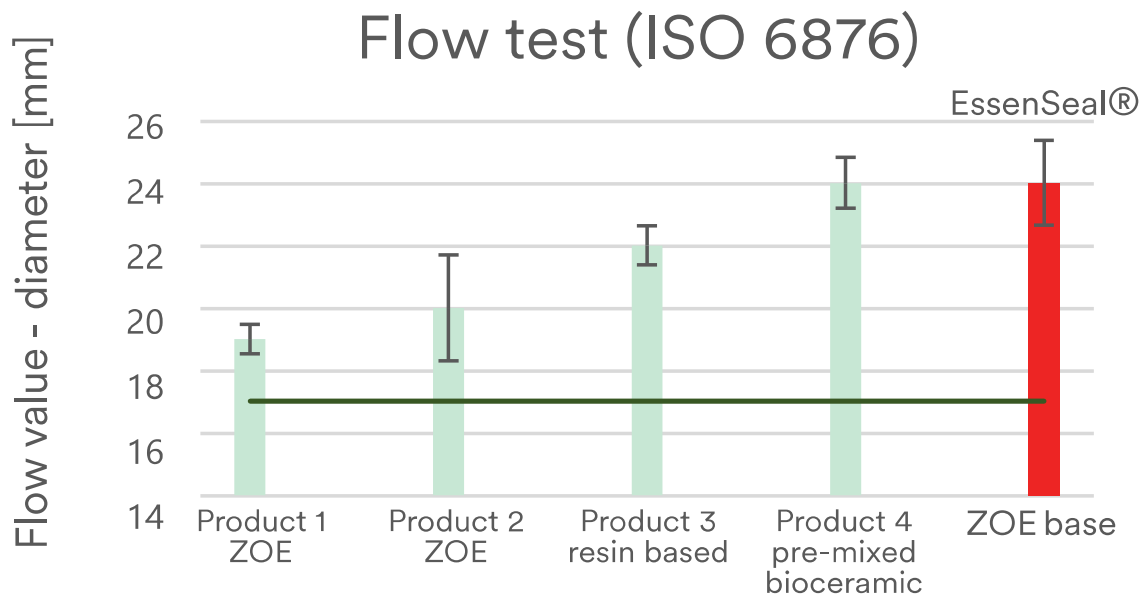

Internal test realized by Produits Dentaires SA

— ISO 6876 requirements (>17mm)

EssenSeal® provides excellent anatomical adaptation thanks to the combination of its thixotropic properties, small particle size, and the consistency induced by essential oils. Accordingly, EssenSeal's flowability facilitates its placement and penetration in the root canal anatomy. The result is a high coating ability and a highly tight obturation. EssenSeal® is used in combination with gutta-percha points for both warm vertical technique and cold lateral technique.

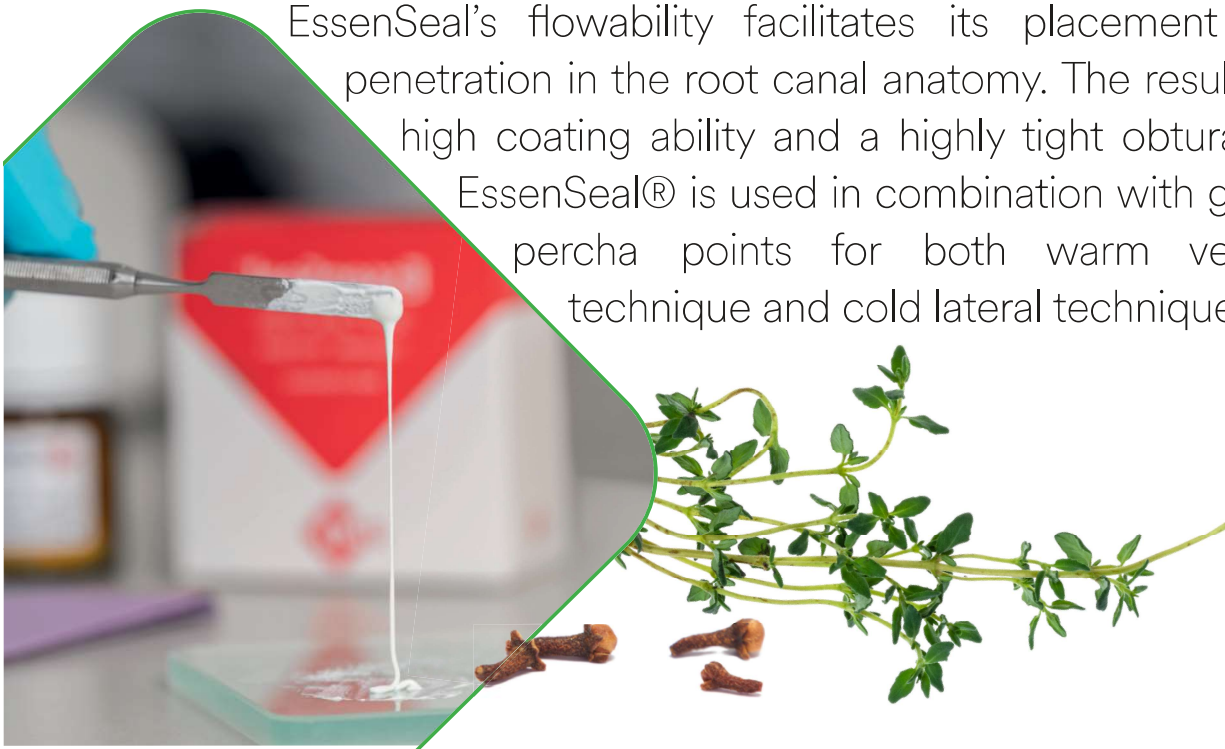



Highly radio-opaque

For precise follow-up • 6mm AI according to ISO 6876




Creamy and smooth consistency

Thanks to fine particles granulometry • The predominant particle size is measured at 7 μ m by laser granulometry.



Optimized clinical workflow

Working time 60 minutes and setting time lower than 3 hours.

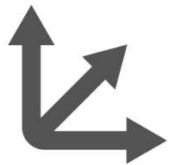

Three-dimensional and tight sealing

with gutta percha points



Find out more about EssenSeal®
and why it is appreciated by
endo lovers globally






EssenSeal®

Complementary Products Gutta-Percha Points

Packaging	Box includes 15 g powder and 10 ml liquid <i>*enough quantity to seal more than 60 root canals</i>
REF	10246
Recommended retail price	Find all of our prices on our website www.pd-dental.com

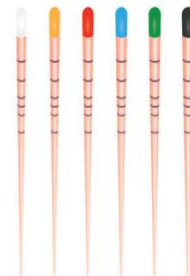

For more PD products check out our catalog
by scanning the QR code



Produits Dentaires SA
Rue des Bosquets 18
1800 Vevey, Switzerland